- Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.
- Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back.

Get the Blood Drop - Palm

ALTERNATIVE SITE TESTING (AST) PALM

Use the **clear AST endcap**.

- Insert the lancet as described in *Prepare the Lancing Device*.
- Attach and lock the **clear AST endcap**. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

CAUTION: Do not use the clear AST endcap on your fingertips.

- Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.

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

- Pull back the re-set button to prepare the lancing device. The release button is raised when the lancing device is ready.
- Press the lancing device firmly against the puncture site, then press the release button UNTIL THE DEVICE CLICKS.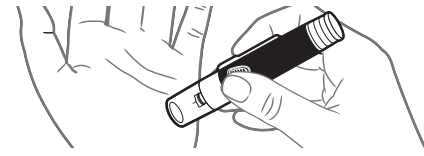

(If the device does not puncture, reset by repeating Step 4.)

- Do not take the device away from your palm. Maintain steady pressure until a small, round blood drop forms under the clear endcap.

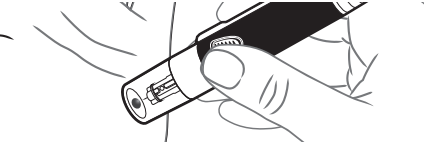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⚠️ WARNING


- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

Do not use AST under the following conditions:

- If you think your blood sugar is low (hypoglycaemia).
- When blood sugar is changing rapidly (after a meal, insulin dose, or exercise).
- If you are unable to feel symptoms of low blood sugar (hypoglycaemic unawareness).
- If you get alternative site blood sugar results that do not agree with how you feel.
- During illness or times of stress.
- If you will be driving a car or operating machinery.

POTENTIAL BIOHAZARD

- The MICROLET NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.
- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- Always wash your hands with soap and water and dry them well before and after testing or handling the meter, lancing device, or test strips.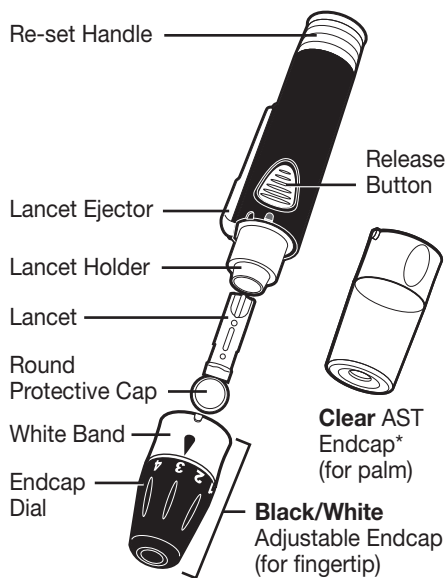


* May not be included in all kits. Available on request from **Customer Service**.

Prepare the Lancing Device

Warning: Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.

- Wash hands and the puncture site with soap and warm water. Rinse and dry well.
- Unlock the endcap. Twist the white band so the white raised mark and black indented mark line up as shown.
- Pull off the endcap.

- Loosen the round protective cap on the lancet by rotating the cap ¼ turn, but **do not remove the cap**.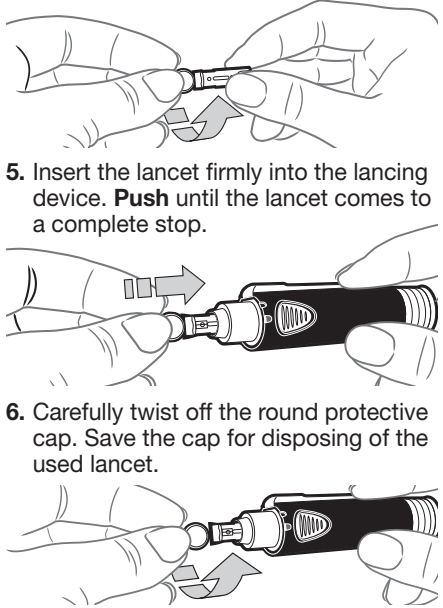


Get the Blood Drop - Fingertip FINGER TIP TESTING

Use the **black/white** adjustable endcap. Do not use the **clear** AST endcap.

- Attach the **black/white** adjustable endcap.
- Lock the endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.
- Rotate the endcap dial to adjust the puncture depth. The amount of pressure applied to the puncture site also affects puncture depth.

- Stroke your hand and finger toward the puncture site to form a drop of blood. Do not squeeze around the puncture site.
- Test immediately after you have formed a small, round blood drop, as shown.
- Wipe the puncture site with a clean, dry tissue and **maintain pressure** on the site until the bleeding stops.

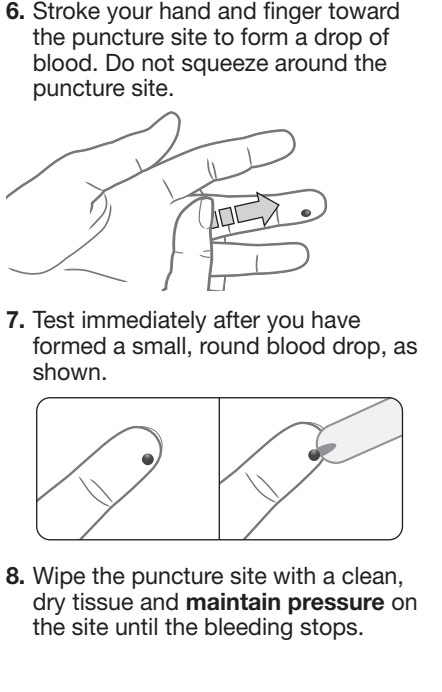Eject / Dispose of the Lancet

Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.** Your MICROLET NEXT lancing device has a lancet ejector. Do not use your fingers to remove the lancet from the lancing device.

1. Carefully remove the endcap from the lancing device. Twist the white band of the endcap so the white raised mark and the black indented mark line up. Pull off the endcap.
2. Place the round protective cap that you saved onto a flat surface. With the used lancet still in the lancing device, push the lancet into the center of the cap.

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